

LESS IS MORE!

- **LESS TO DO**

Fewer steps to accomplish the same procedures

- **LESS TO BUY**

Fewer accessories needed for additional functionality

- **LESS TO PAY**

Lower initial costs than competitors with lower priced accessories

MORE USER FRIENDLY



Change Power Advantage – Less To Do...

The Aaron 900™ allows the doctor to change the power setting with one action, using the rotary power-control dial or the up/down buttons on the autoclavable handpiece.

Here are the steps necessary just to go from Low to High Power on the Hyfreator® 2000.

(reverse steps to go from high to low power)



Step 1

Unplug the handpiece from the Lo power receptacle



Step 2

Move the power selector switch from Lo to Hi



Step 3

Plug the handpiece into the Hi power receptacle



Step 4

Adjust the rotary dial or up/down buttons on the handpiece

Foot Control Advantage – Less To Buy...

The Aaron Way



Simply plug in the footswitch and you are ready to go!



When compared with the Aaron 900, the Hyfrecator® 2000 costs nearly twice as much for the foot control feature. However, the handpiece power control feature is lost, as well as button activation.

Additionally, the Hyfrecator's setup is much more complicated. With the Aaron 900, just plug in the footswitch and you are ready to go. You maintain power control from the handpiece, as well as activation from the handpiece or foot control; it's your choice.

Don't pay more for less. The Aaron 900 is the better choice.

The Conmed® Way



The Clear Choice is Aaron.

Here are the steps necessary to go from 3-button handpiece control to foot control on the Hyfrecator 2000. (steps must be reversed to go back to 3-button handpiece control.)



Accessory Advantage – Less To Pay...

FOOTSWITCH

A803 Footswitch, 1/box

HANDPIECE

A901 Autoclavable reusable 3-button handpiece, 1/box

A910 Handpiece Sheath, 100/box, Non-sterile

ELECTRODES

A804 Dermal Tips, Sharp, 100/box, Non-sterile

A805 Dermal Tips, Sharp, 50/box, Sterile

A806 Dermal Tips, Blunt, 100/box, Non-sterile

A807 Dermal Tips, Blunt, 50/box, Sterile

A811 Reusable Electrode, 1/box

A830 Angled Sharp Electrode, 1/box

A831 Angled Ball, 1/box

A832 Straight Ball, 1/box

A833 Straight Needle, 1/box

A834 Epilation Needle, 1/box

A835 Extended Straight 5 mm Ball, 1/box

A836 Extended Angled Sharp Electrode, 1/box

A802EU Dispersive Electrode

FORCEPS

A820 7.25" Bayonet Fine, Smooth, 1/box

A821 8" Bayonet Medium, Smooth, 1/box

A822 7" Straight Fine, Smooth, 1/box

A823 7" Curved Fine, Smooth, 1/box

A824 4.5" Curved Fine, Smooth, 1/box

A825 4.75" Straight Medium, Smooth, 1/box

A826 4.5" Straight Fine, Smooth, 1/box

A840 4" Tenzel with 5 mm Tip - Coated, 1/box

A841 3.5" McPherson Curved with 5 mm Tip - Coated, 1/box

A842 3.5" McPherson Straight with 5 mm Tip - Coated, 1/box

A843 4" Tenzel with 5 mm Tip - Uncoated, 1/box

A844 3.5" McPherson Curved with 5 mm Tip - Uncoated, 1/box

A845 3.5" McPherson Straight with 5 mm Tip - Uncoated, 1/box

A827EU Forceps Cord, 1/box

MOUNTING OPTIONS

A812-C Mobile Stand with Bottom Tray, Top Tray, Clamp, and Height Extension

A812 Mobile Stand


A808-T Top Tray for A812

A812-BT Bottom Tray for A812

A812-E Height Extender for A812

A813 Table Top Stand

A837 Wall Mount Kit

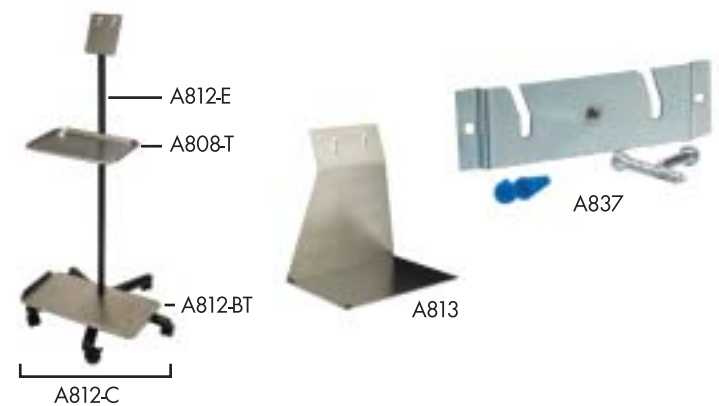

Technology Advantages – Less To Worry About...

The Bovie 3-Button Handpiece – Better By Design, Inside And Out

The basic function of a 3-button handpiece is to allow the user to control power up/down and to activate the high frequency desiccator. The Aaron 900 has a technology advantage right in the handpiece: a superbly engineered collet that's even gold plated for superior conductivity. The collet is attached to our circuit board and the entire handpiece is encapsulated and formed in a high tech polymer that combines comfort for you with a sure grip. The buttons on the handpiece have been designed for superior tactile feel and comfort. Finally, it's autoclavable so you can have sterility when you need it, and an easy to clean and durable handpiece all the way around.

Digital Error Detection

Digital error detection means unsurpassed safety for you and your patient. The Aaron 900 continuously monitors every aspect of the unit's output. At the sign of any problem the machine instantly disables the output and displays the appropriate error code on the display.



SPECS

- Output Power 0-10 Watts adjustable in 1/10 Watt increments
10-30 Watts adjustable in 1 Watt increments
- Line Voltage A900 (A800EU-110) – 120 VAC ± 10%
A900-220 (A800EU-220) – 220 VAC ± 10%
- Output Frequency 550 kHz
- Line Frequency 50-60 Hz
- Dimensions Width = 22.2 cm (9") • Height = 15.9 cm (6.25")
Depth = 11.4 cm (4.5") • Weight = < 2.3 kg (< 5 lbs.)
- Warranty One Year (unit only)

Designed For Years of Trouble-Free Use

The Aaron 900 is designed and engineered to be the most reliable and durable desiccator available today. The unit's life is prolonged by incorporating the latest in chip technology and reducing the number of internal parts, thereby generating less internal heat. Post-procedure and long-term cleanup are simplified by its non-absorbing, nonstaining, high-impact plastic case.

Package



Unit, power cord, handpiece (A901), 50 non-sterile disposable sharp dermal tips (A804), 50 non-sterile disposable blunt dermal tips (A806), wall mount kit (A837), and user's guide.

A Lower Cost Alternative

If cost is a major factor, consider the Aaron 800-EU. You get the same great technology, just without the 3-button handpiece and with fewer included electrodes. It costs around \$100 less.

Package includes unit, power cord, handpiece, one non-sterile reusable electrode, wall mount kit, and user's guide.



DISTRIBUTED BY:

CAPITAL MEDICAL SUPPLIES PTY LTD
71 HEFFERNAN STREET
MITCHELL. ACT.2911
PH. 02 62415511
FAX. 02 62417538
E-MAIL. info@capitalmedicalsupplies.com.au



7100 30th Ave. N. • St. Petersburg, FL 33710-2902
U.S. Phone 1-800-537-2790 • Fax 1-800-323-1640
Int'l. Phone +727-384-2323 • Fax +727-347-9144
www.aaronmed.com • sales@aaronmed.com